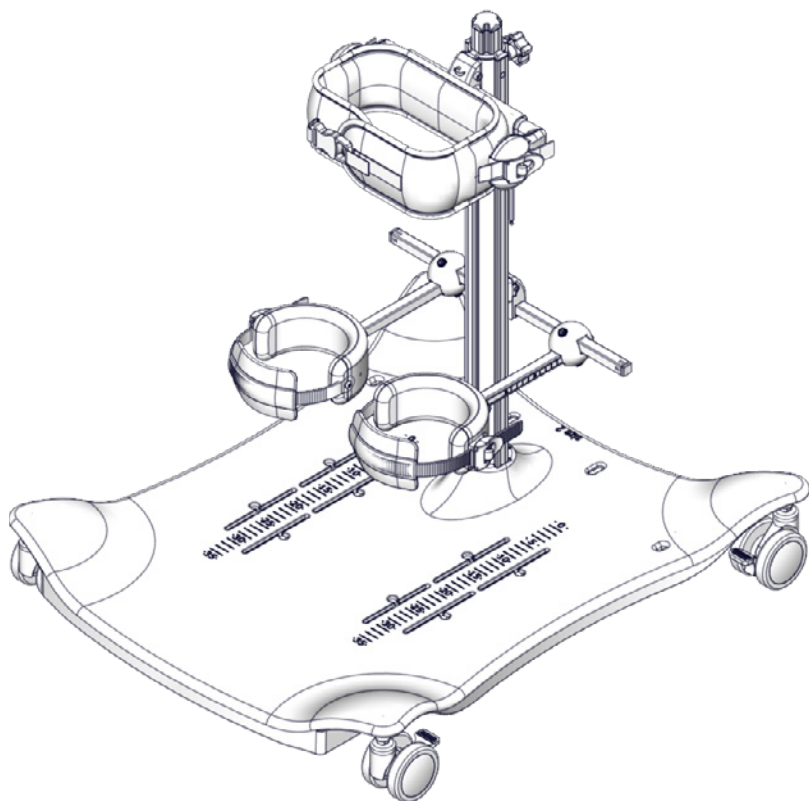
User manual

Meerkat

R82[®]
by Etac


M1039 - 2024-01-24 - Rev. 10.0


www.etac.com


	3	ENGLISH DEUTSCH	29 - 32 33 - 38
	4 - 6	NEDERLANDS DANSK	39 - 43 44 - 47
	7	NORSK SVENSKA	48 - 51 52 - 55
	8 - 9	SUOMI ESPAÑOL	56 - 58 59 - 63
	10 - 11	FRANÇAIS ITALIANO	64 - 68 69 - 72
	12	PORTUGUÊS PORTUGUÊS DO BRASIL	73 - 77 78 - 81
	13	РОССИЯ POLSKI	82 - 87 88 - 92
	14	ČESKY 中國	93 - 96 97 - 100
	15	日本 ΕΛΛΗΝΙΚΗ	101 - 106 107 - 112
	16 - 26		
mm (inch) kg (lb)	27		
	28		

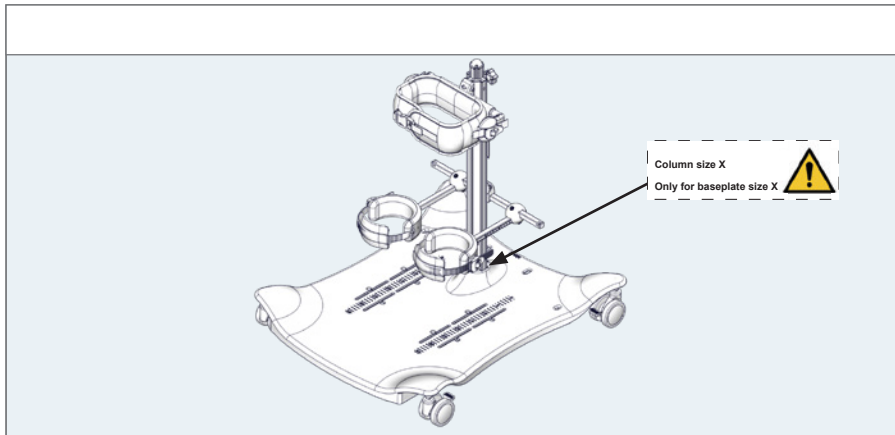
MEERKAT 1

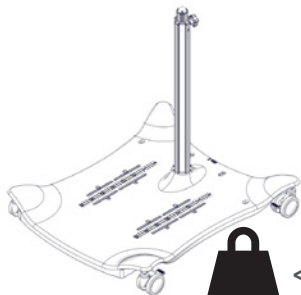MEERKAT 2

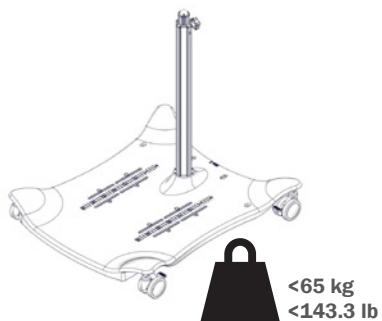
MEERKAT 3






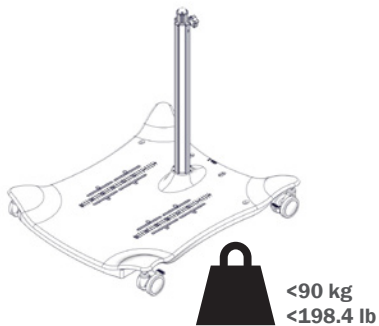
 02

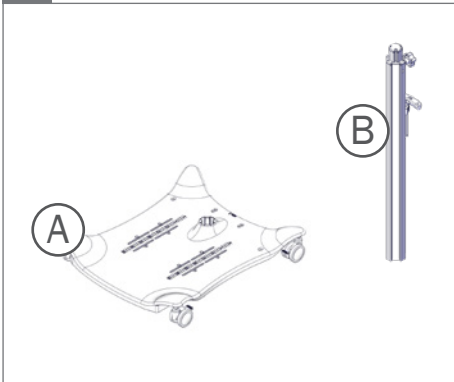
A\B	1	2	3
1	✓		
2		✓	✓

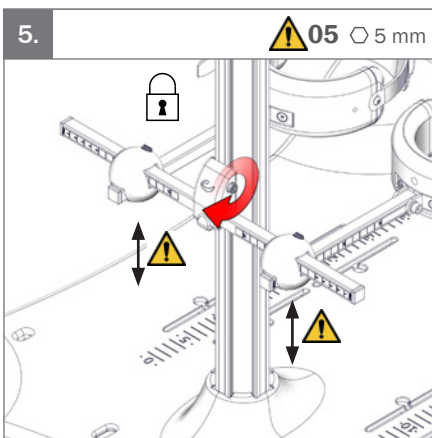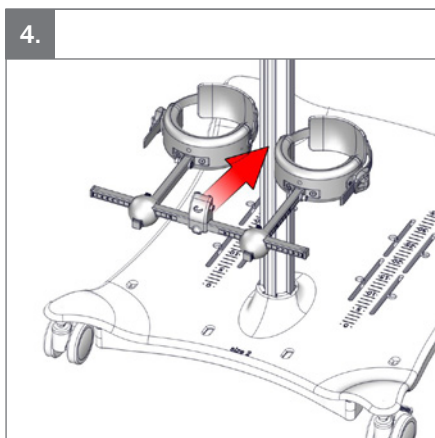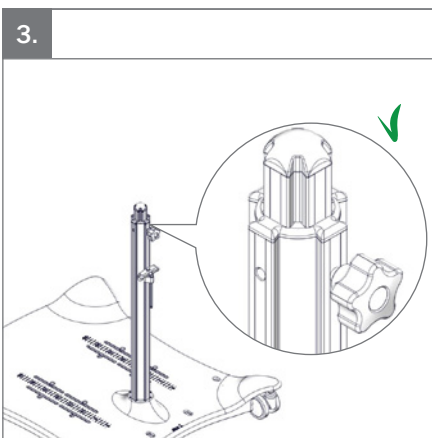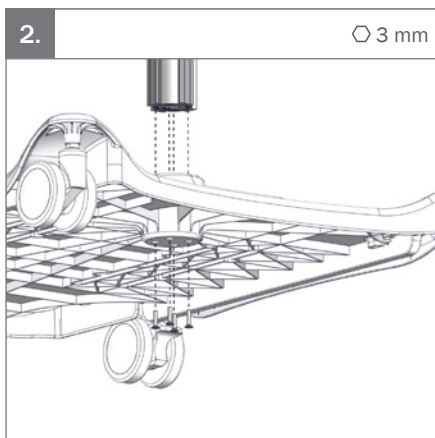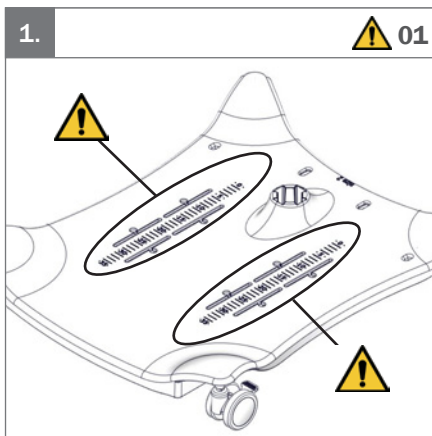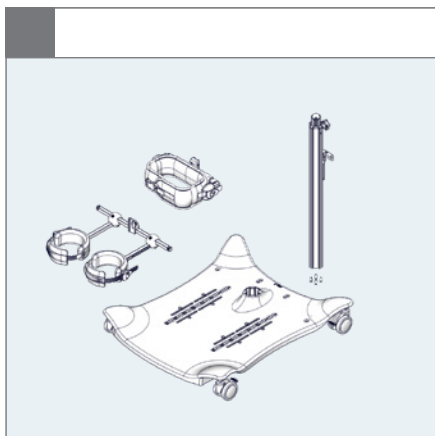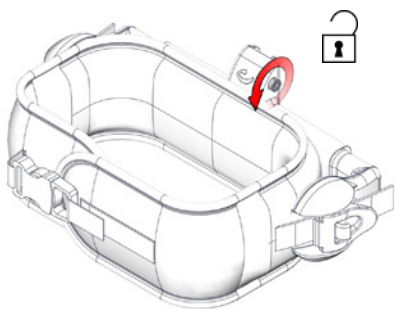
6.

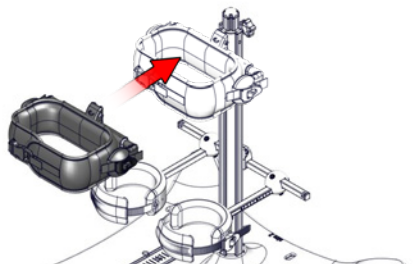
7.


8.

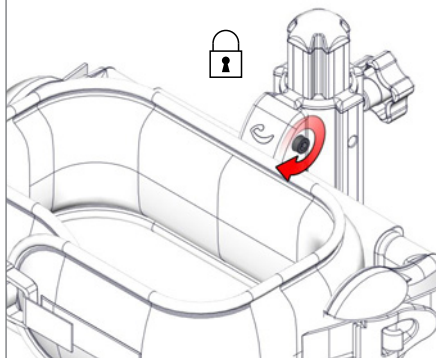○ 5 mm

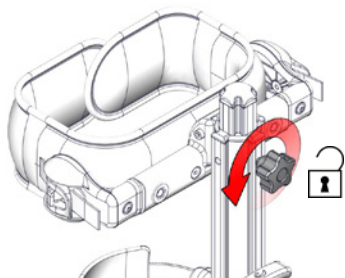
9.


10.

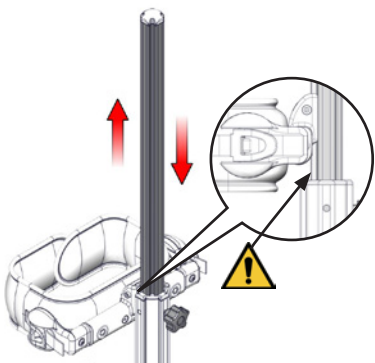⚠ 05

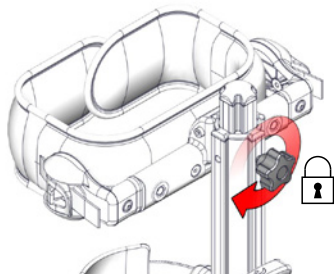
11.



  05

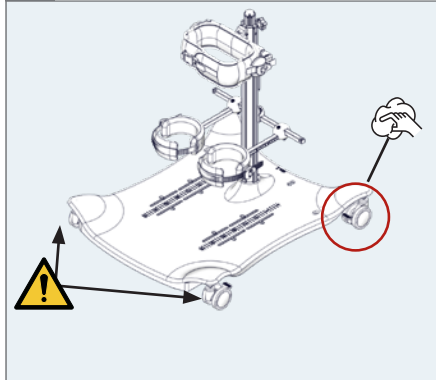
1a.

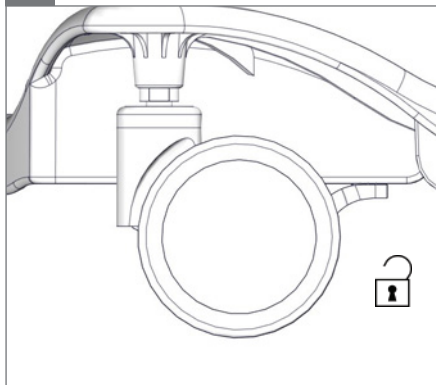1b.












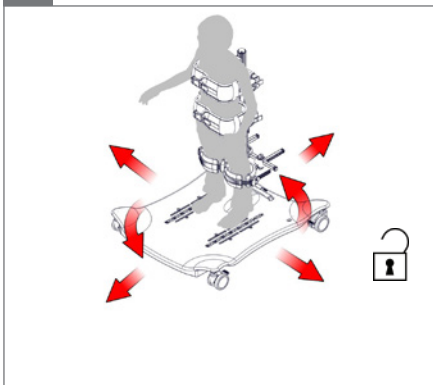
2a.










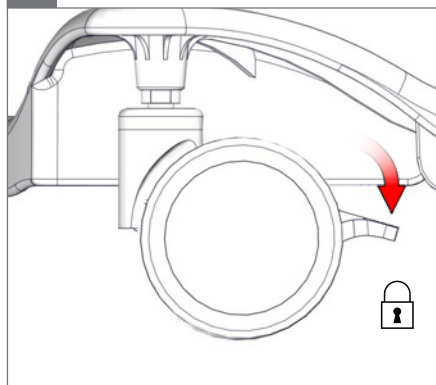
2b.

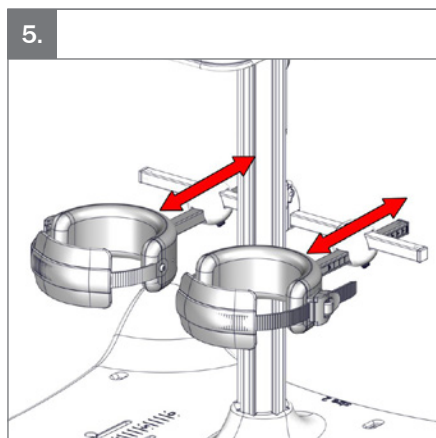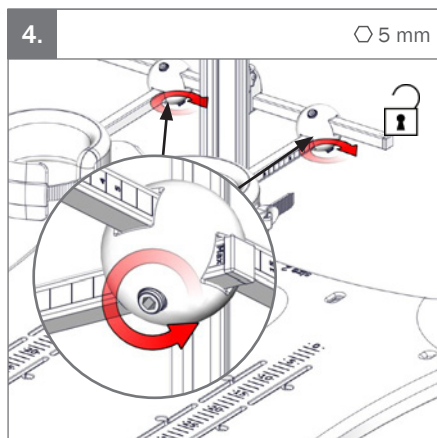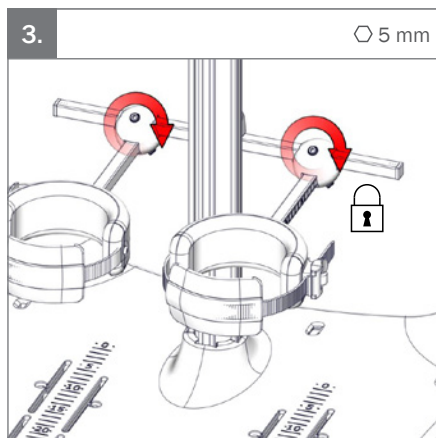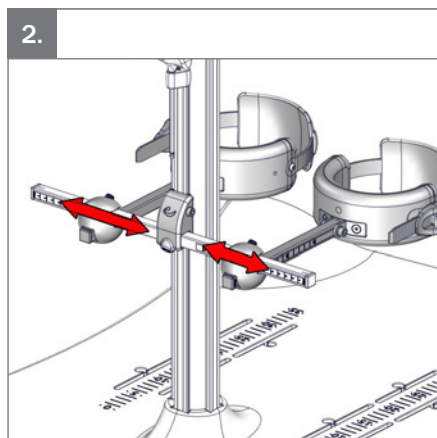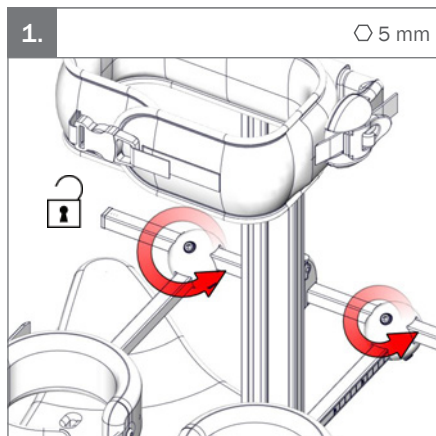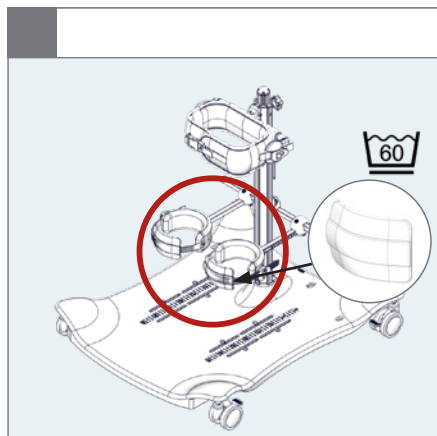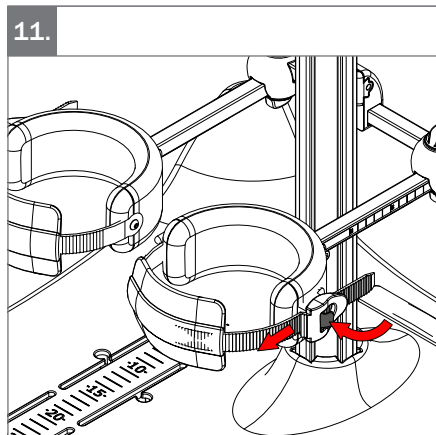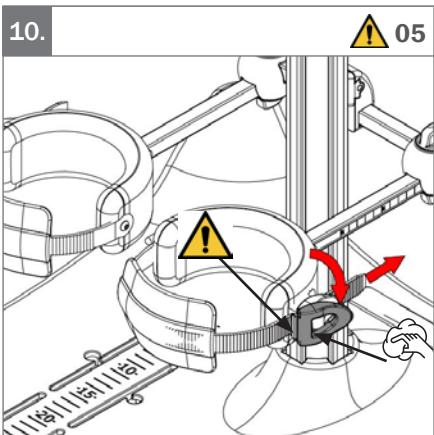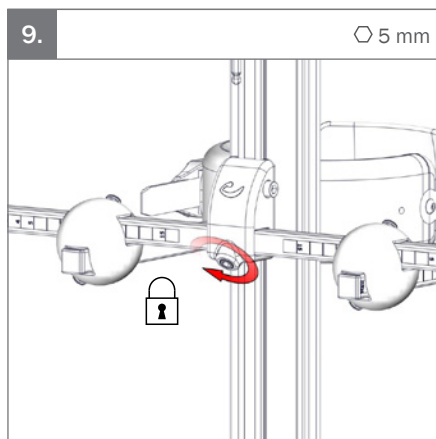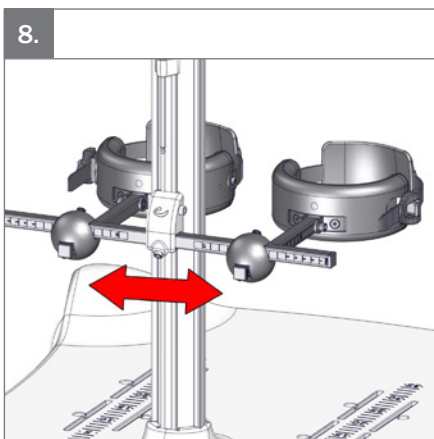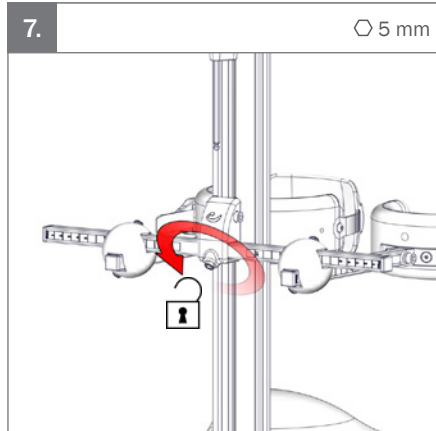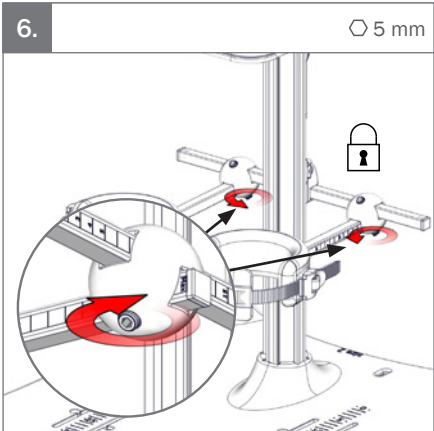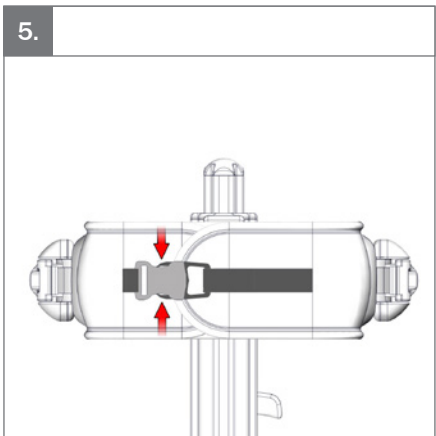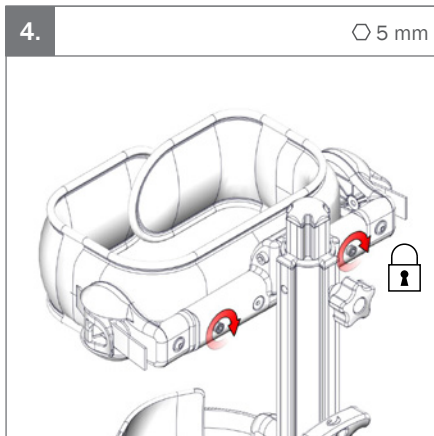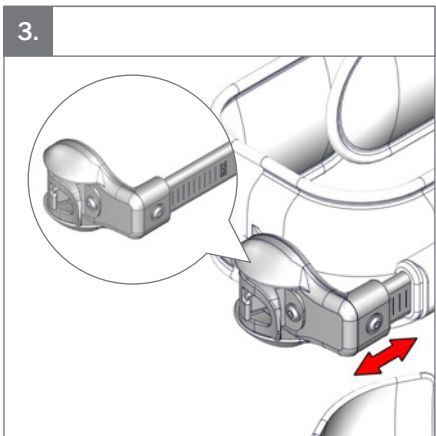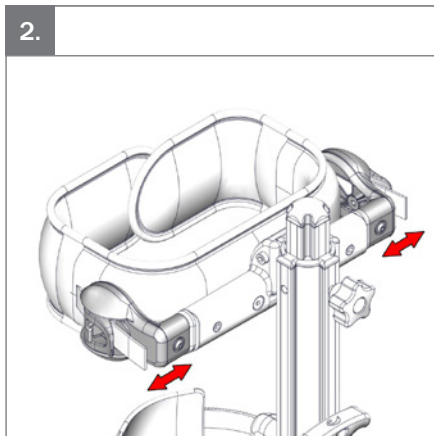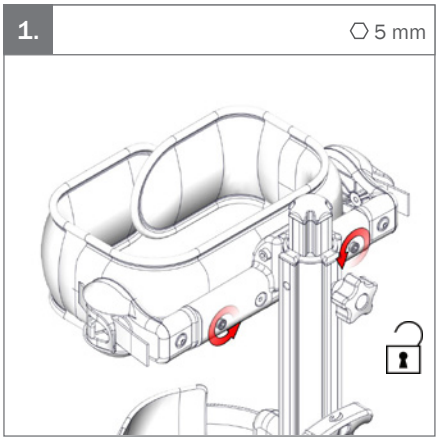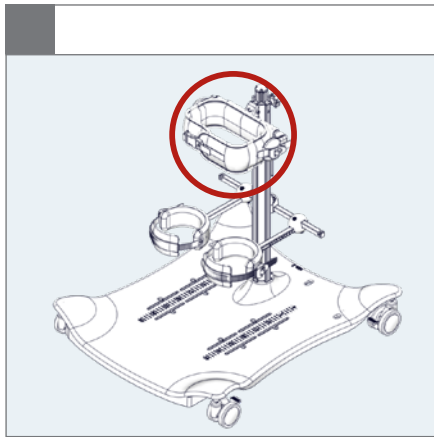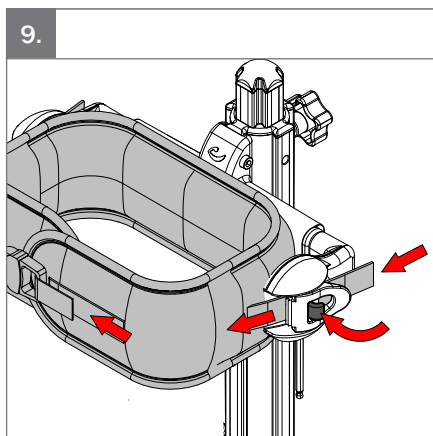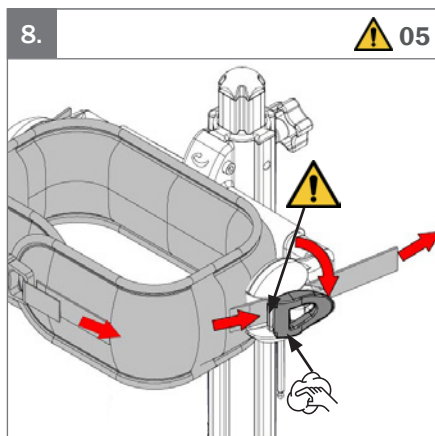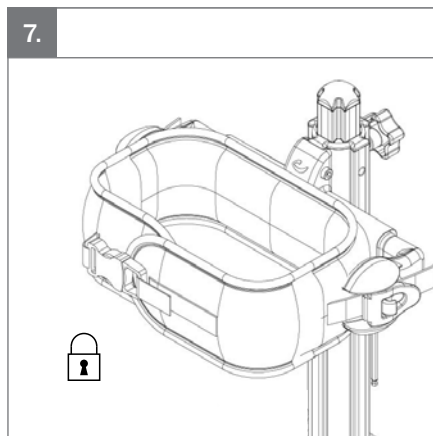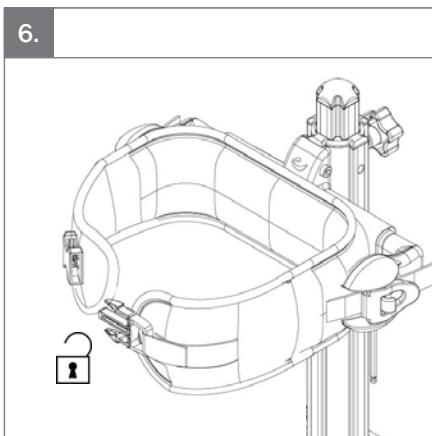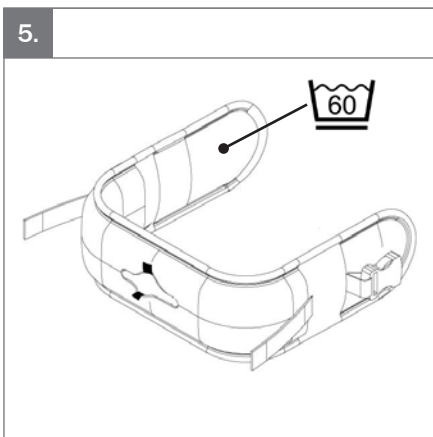
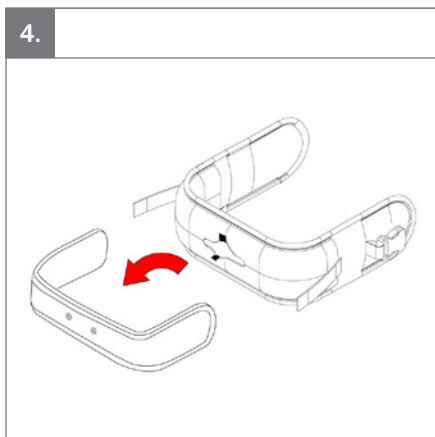
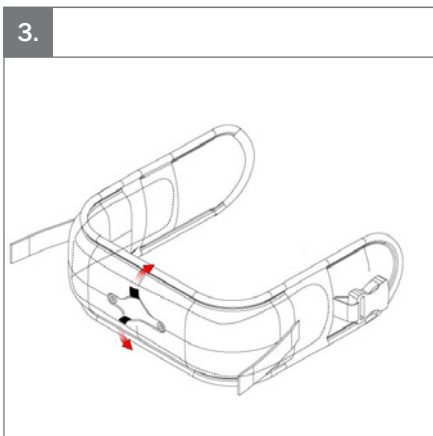
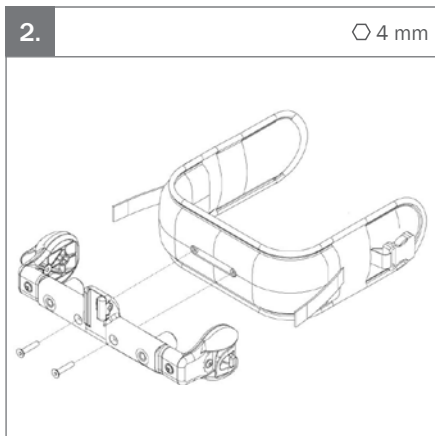
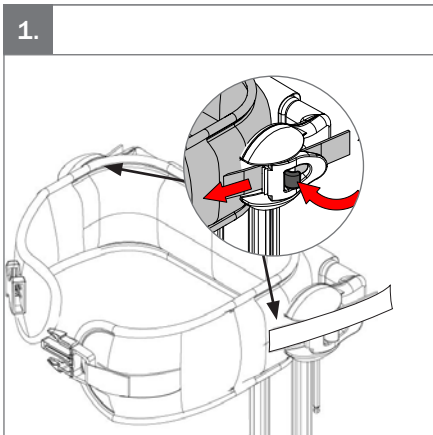
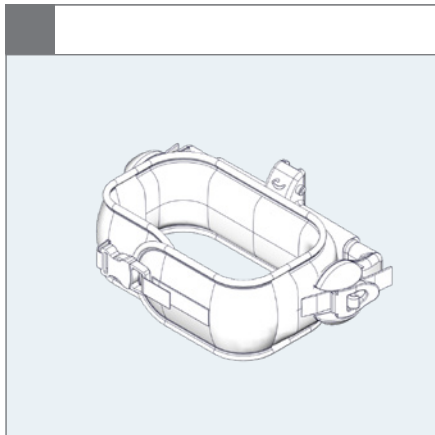
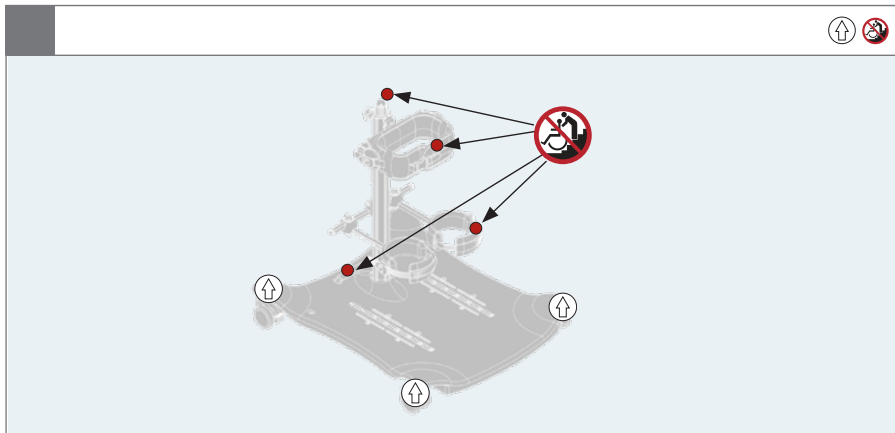
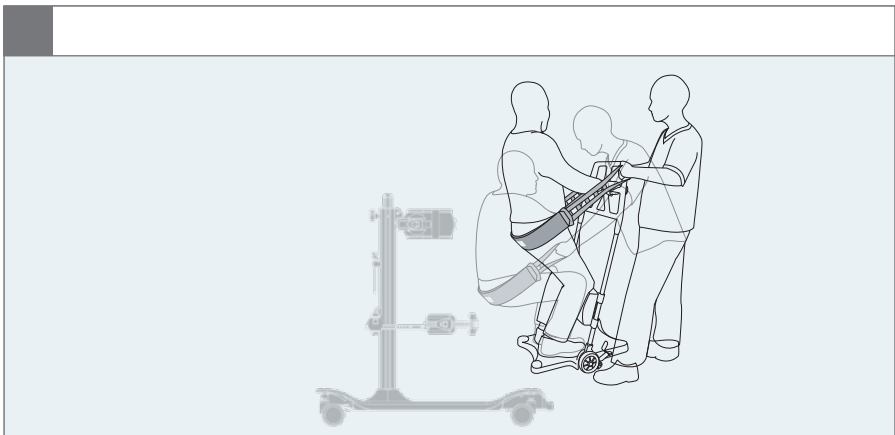
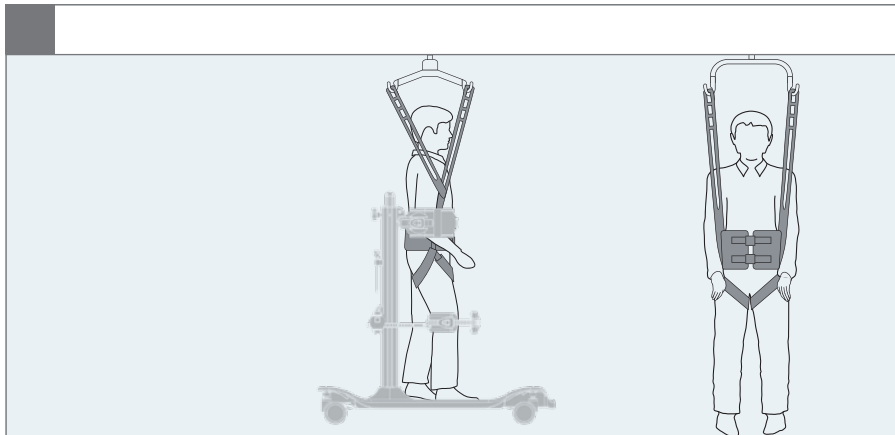
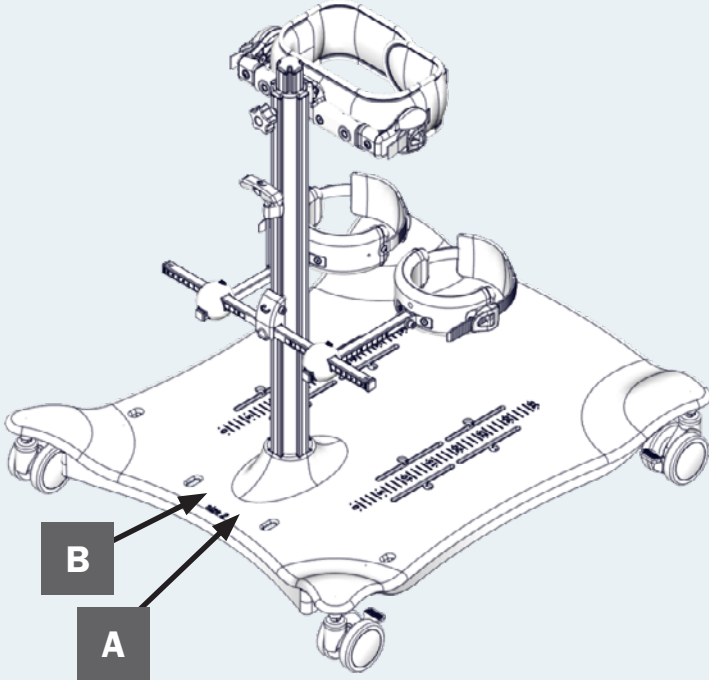
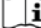
A

R82 A/S, Parallelvej 3
 DK-8751 Gedved
 etac.com
 Made in xxxxxxxx

YYYY-MM-DD

B

Posting group
 Product name, size x

DMR ver.: xxxxx ▲ Max. user mass: xxx kg
 SR xxxxxxxxxxxx ▲ Max. rated load: xxx kg
 REF xxxxxxxxxxxxxxxx ▲ Mass device: xxx kg

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
Accessories

	S	M	L	XL	XXL
Hip support, circumference	450–600 mm	600–700 mm	700–900 mm	900–1100 mm	1000-1200 mm
Chest support, circumference	450–600 mm	600–700 mm	700–900 mm	900–1100 mm	1000-1200 mm
Hip and chest supports, width	190–205 mm	255–335 mm	255–335 mm	-	-
Sandals heel, inside width	60–75 mm	65–80 mm	85–95 mm	-	-
Sandals forefoot, inside width	95–110 mm	100–145 mm	120–165 mm	-	-
Knee supports, circumference	189–251 mm	251–290 mm	290–390 mm	-	-
Knee supports, inside width	60 mm	80 mm	107 mm	-	-

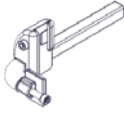
9996097320

9996097350

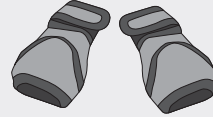9996097321


9996097345

9996097323

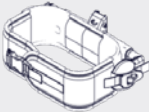N/A

9996097324

N/A

9996097325

N/A

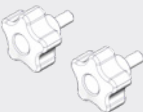
9996097326

N/A

9996097327

N/A

9996097328

N/A

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.

Intended purpose

The device is a standing frame for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who experience limitations in their ability to maintain a standing position. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age. The user is the person standing in the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down or sitting in another assistive device or a conventional product.

Contra indications

There are no known contra indications.

Precautions

Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories.
- Movement function (e.g. experience involuntary movements).
- Ability to maintain a standing position, use the product with accessories (e.g. chest or back support).
- Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head.

Application areas

The product is for indoor use

Enclosed tools:

- 3+4+5 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). The CE mark must be removed when rebuild Further, this product conforms to the requirements according to:

- EN 12182

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the user manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide.
- Users who have difficulties reading the user guide can have someone read the user guide aloud to understand how to use the product.
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that the user is supported, when performing any adjustment on the product or accessories

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to

measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- As a minimum, use the product with a support placed at the top of pelvis or higher
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82: Cushioning
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

Accessories

Rocker

- Can damage hard and soft surfaces

Daily adjustments

- Hip/chest supports are adjustable in circumference and distance to the floor, using the ratchets on the supports and the knob on the center column.
- Knee supports are adjustable in circumference using the ratchets on the supports. Furthermore the knee

extension/flexion level is adjustable with the daily adjustment kit offered as an accessory to the product.

- All other adjustments can be done by using the Allen key supplied with the product.

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Fittings and buckles for signs of worn out parts.
- Castors are moving freely and all wheel locks/brakes are easily activated.
- Nuts and bolts on the product are tightened.
- Product for cracks or signs for worn-out parts.
- All labels on the product are intact.
- Check that all castors are moving freely and that all wheel locks/brakes are easily activated

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Plastic
- Rubber
- Steel
- Fabric

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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Etac Ltd.
Gelbart AG