### E-Z TURN II DISC

Reduce caregiver injuries - Safe and secure patient transfer - Reduces risk of patient falling. The EZ Turn Disc provides a safe and easy patient transfer from one seated position to another. The brake pedal can be pressed down to keep the disc from acting like a "Lazy Susan", so that the patient can be slowly swiveled to the desired position. The knee pads pivot to better fit each individual patient and are adjustable to 3 settings for width between the knees. The vertical support bar is adjustable allowing for correct user height. Maximum weight capacity: 300 lbs (136 kg). Unit weighs 46 lbs (20.4 kg).

Infection Control: How to properly clean the hand grips on the EZ Turn - You can use any cleaning product - the manufacturer does not recommend a specific one. The material used on the handles is closed cell and will not absorb anything into the surface - it is unaffected by most solvents and chemicals. The material used on the handles is non-absorbent but it is porous so the manufacturer recommends avoiding cleaners that are grainy (such as Comet) because granules may get stuck in the porous surface. For cleaning and disinfecting in a hospital or care facility setting, an accelerated hydrogen peroxide can be used to clean the handle.

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### Limited One Year Warranty

Parsons ADL warrants to the original owner of this product that it is free from defects in material and workmanship for one year from date of purchase. This warranty is offered provided the product has not been abused, misused, altered or damaged after purchase. (This includes damage due to the use of harsh chemicals.)

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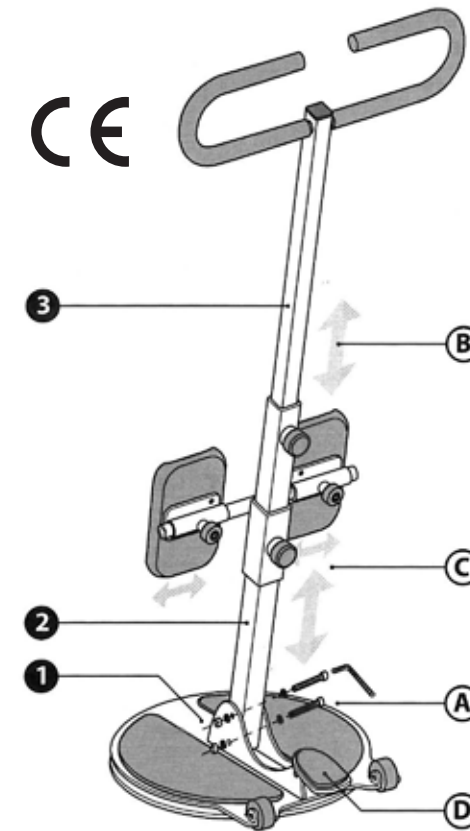
Figure 1

#### Assembly Instructions (See Figure 1):

Remove bolt securing main tube (2) to base (1).

Align base (1) and main tube (2). Assemble with bolt (A).

#### Instructions For Use (See Figure 2, 3 & 4):

Safe and easy patient transfers from one seated position to another seated position with the E-Z Turn Disc:

- Adjust the height of the handle (3) to suit the patient and caregiver (B).
- Position the E-Z Turn Disc between the seated patient and the other seated location.
- Adjust the height and width of the knee pads (C). Position the patient's knees centrally on the knee pads.
- The caregiver locks the brake pedal (D) and securely holds the upper handle bars.
- The patient grabs the lower handle bar and pulls themselves to an upright position (Figure 2).
- While the patient securely holds the lower handle bar, the caregiver releases the brake, rotates/swivels the E-Z Turn Disc (Figure 3).
- The caregiver locks the brake and the patient lowers themselves to the new seated position (Figure 4).

⚠ Built-in wheels make the E-Z Turn portable and maneuverable, but are not intended for moving patients during transfer.

Figure 2

Figure 3

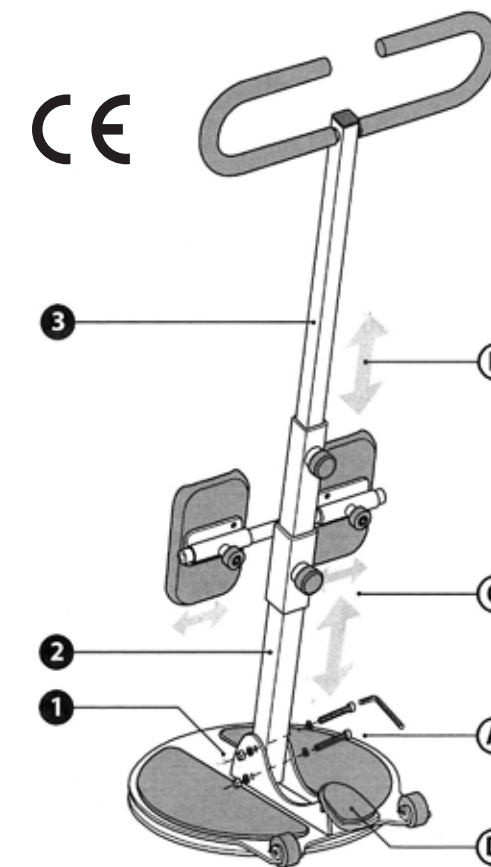
Figure 4

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Parsons ADL Inc.