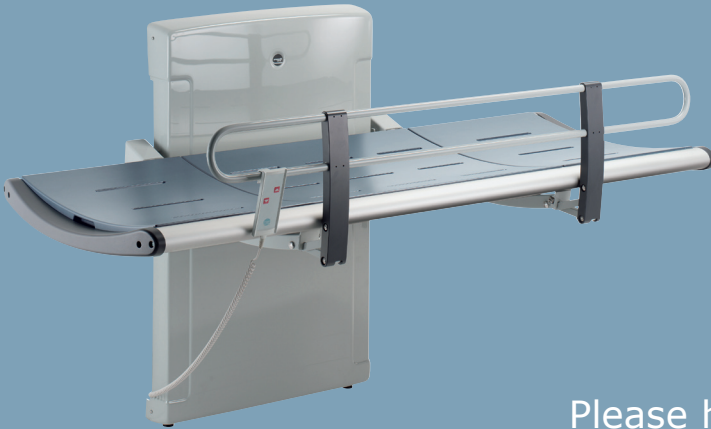
keep living

Please hand to user

- UK:** Nursing Bench
Operating and maintenance manual
- DK:** Bruseplejebriks
Brugs- og vedligeholdelsesvejledning
- NO:** Pleiebenk
Bruks- og vedlikeholdsveiledning
- SE:** Duschbriks
Bruks- och underhållsanvisning
- DE:** Dusch- und Pflegeliege
Gebrauchs- und Wartungsanleitung
- FR:** Table de soins pour douche
Manuel d'entretien et mode d'emploi
- ES:** Camilla sanitaria
Manual de uso y mantenimiento
- NL:** Verzorgingsbank
Bedienings- en onderhoudshandleiding

R8503 R8508
R8510 R8511
R8513 R8515
R8516 R8518
R8519 R8520
R8523 R8528
R8533 R8538

English.....	3
Dansk	7
Norsk.....	11
Svenska	15
Deutsch	19
Français	24
Español.....	29
Nederlands	34

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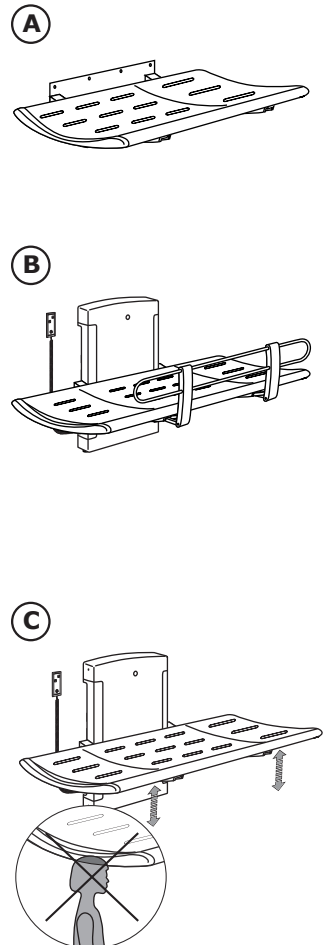
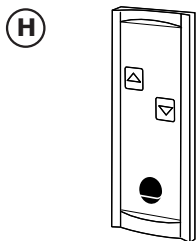
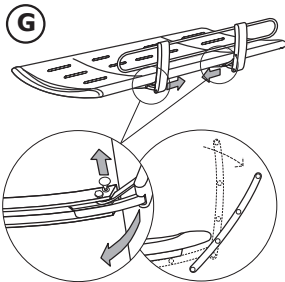
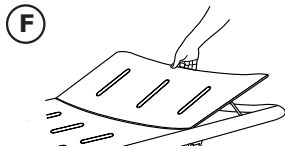
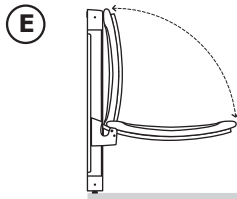
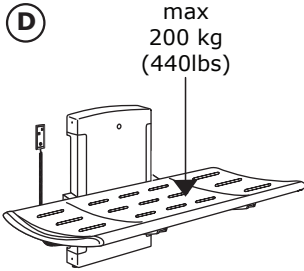
Thank you for purchasing this Pressalit Care nursing bench. This product is intended to use for bathing a person in a supine position. We hope you will benefit from the many features:

- Comfortable surfaces – the bench, end caps and safety rail brackets are covered with polyurethane foam.
- The slightly curved form makes the bench extremely comfortable and safe.
- An integral water collection tray and outlet limits water splashing on carers and the floor.
- A gas cylinder enables the bench to be folded easily against the wall when not in use to maximize floor space.
- The manually adjustable back support can be used at either end for increased comfort.
- Pressalit Care manufactures fixed height nursing benches and adjustable height nursing benches with electric motors available with two different surface lengths (1300 mm and 1800 mm). Both sizes are available with or without a safety rail. The figure **(A)** shows a fixed height nursing bench, 1300 mm, without safety rail, while figure **(B)** shows an adjustable height nursing bench, 1800 mm with safety rail.

Important safety information

- Please read and understand the operating and maintenance manual before using the nursing bench. We take no responsibility if the product is used any other way than specified in this manual.
- The information in this operating and maintenance manual is based on the understanding that the nursing bench is correctly installed according to our mounting instructions.
- The product is not intended to be played with.
- If the product is out of order or damaged, it must not be used until it is repaired.
- The content of this manual is subject to alteration without further notice.
- For safety, nothing should stand in the way of the nursing bench when in motion **(C)**.
- We recommend that children do not operate the nursing bench unsupervised.

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When the weight is evenly distributed on the surface, the maximum user weight is 200kg for all models.

Before initial operation

- Check that the voltage of the nursing bench (model with electric motor) corresponds to the mains supply being used.
- Pressalit Care offers a number of different nursing benches and the installation instructions can vary. The information in this operating and maintenance manual is based on the understanding that the nursing bench is correctly installed.

Using the nursing bench

- When unfolding the nursing bench **(E)**, make sure to support the nursing bench with your hand or body during the process, otherwise you may risk damaging the unit.
- The smaller end section of the bench mattress can be used as a head and neck support. Pull the mattress section out of the mattress frame and adjust the height by positioning the metal support struts **(F)**.
- To position the safety rail for use, simply push down against the rail to unlock it from the underside of the bench and swing the rail up and towards you. The safety rail will automatically engage into its locked position.
- To fold the safety rail out of the way, the two locks must be released at the same time **(G)**, then swing the rail towards the underside of the bench.
- Push the rail against the underside of the bench to lock into place.

Adjusting the product

- When transferring a user to the height adjustable nursing bench, you can easily adjust the height of the bench by using the remote control **(H)**, which has two indicators. By pressing and holding the "up" indicator, the nursing bench will rise up slowly. Let go of the indicator to stop the bench. Press and hold the "down" indicator to lower the bench, and again let go of the indicator to stop the bench.

- When mounted at the recommended height, the bench can be adjusted from 300 to 1000 mm above floor for height adjustable nursing benches with electric motors.
- The recommended bench height is 865 mm above the floor for fixed height nursing benches.

Cleaning

Daily:

- Use ordinary soapy water. Avoid aggressive cleaning agents containing acids or chloride. Rinse off with fresh water and dry with a soft cloth.
- To clean the water collection tray, simply remove the mattress sections (I).
- The product cannot be cleaned in an autoclave.

Occasionally:

- Removing limescale marks on aluminium parts: use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

Maintenance

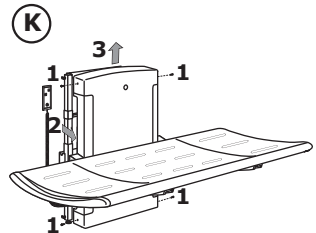
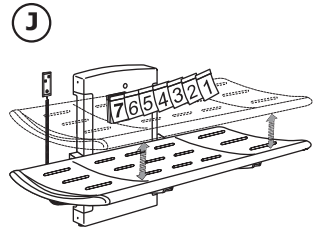
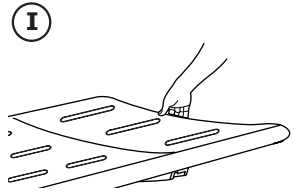
We recommend fully activating the motors used in our height adjustable product at least once every week. This means raising the product to its highest level and lowering it to its lowest level in order to lubricate the internal parts (J).

After-sales inspection

We recommend a yearly inspection carried out by e.g. a contractor.

The procedure is as follows:

- Remove the upper and lower securing screws of the coach housing and carefully remove the housing. The figure (K) explains the 3 steps: 1 remove the screws, 2 open the housing, 3 lift up the housing.
- Check wall mounting fixtures and tighten if required.
- Check all moving parts for wear.
- Check all mountings on nursing bench, safety bar and securing brackets and tighten as required.
- Check waste connections and outlet pipe-work, check and clean trap (where fitted).
- Check all electrical connections to the control box.
- Check isolation fuse (5 Amp) and change if incorrect.
- Test operation of all systems.

- Carefully refit the coach housing and secure.
- Check mattress sections for any damage and clean in accordance with recommendations.
- Issue report of any defects located to client.

Troubleshooting

This chapter summarises the most common problems you may encounter with the nursing bench. If you are unable to solve the problem, please contact Pressalit Care or one of its representatives. Please avoid carrying out repairs yourself as your guarantee does not cover defects that may be caused by carrying out such repairs.

- For models with electric motor, if the nursing bench stops functioning even though the remote control is used correctly, check that there is power and that all wiring and plugs are intact and connected correctly. If there is something wrong or suspicious, please contact a qualified electrician for a further check-up.

CE

 This CE marking shows compliance of this equipment with Council Directive 93/42/EEC of 14 June 1993 concerning medical devices and later amendments.

Information: disposal and recycling of this product

This product must not be disposed of together with ordinary household waste, as electrical and electronic waste must be disposed of separately. In accordance with the WEEE directive, every member state must ensure correct collection, recovery, handling and recycling of electrical and electronic waste. Private households in the EU can take used equipment to special recycling stations free of charge. Contact your retailer, distributor or the municipal authorities for further information on what you should do with electrical and electronic waste.

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Pressalit Care A/S
a division of Pressalit Group A/S