



Wall Mount Electric Height Adjustable Change Table

Model # 19-88-62

Size: 62"x24"

Model # 19-88-72

Size: 72"x24"

INSTRUCTION GUIDE

Read Thoroughly before Operating

Manufactured in Canada by Can-Dan Rehatec Ltd.

3-1378 Sandhill Drive, • Ancaster, Ontario • L9G 4V5

www.can-dan.com • info@can-dan.com • Tel: (905) 648-7522 • Fax: (905) 648-7799

TABLE OF CONTENTS

1. Safety Precautions
2. Scope of Application
3. Product Information / Wall Mount Table
4. Installation Instruction
5. Operation and Handling Instructions
6. Maintenance
7. Trouble Shooting
8. Warranty

Safety Precautions

Before using, always make certain:

- ✓ Care-giver has received appropriate instructions and training.
- ✓ The Instruction Guide has been carefully studied and understood.
- ✓ The individual is secured to the table using the 2 (two) integrated safety belts provided.
- ✓ Do not, at any time, leave anything under the change table

***Requires a GFI receptacle (15 amp)**

***Maximum Lifting Capacity: 250 Lbs.**

NEVER LEAVE USER UNATTENDED ON TABLE

Scope of Application


The Wall Mount Changing Table is a lifting device. Through the use of an electrically powered hand control, which is connected to a 24V actuator, the table is lowered to the height of the wheelchair enabling the caregiver to provide transfer assistance onto the table in a safe manner.

Once on the table, the individual must be secured using the two (2) integrated safety belts. The caregiver can then proceed to raise the table to their ideal working height.

The Wall Mount version of the table comes equipped with (3) three wire baskets which can hold accessories required for changing and cleaning.

With a lifting capacity of 250 lbs., it is suitable for use in schools, clinics, private homes, and other care-giving situations for children.

Product Information

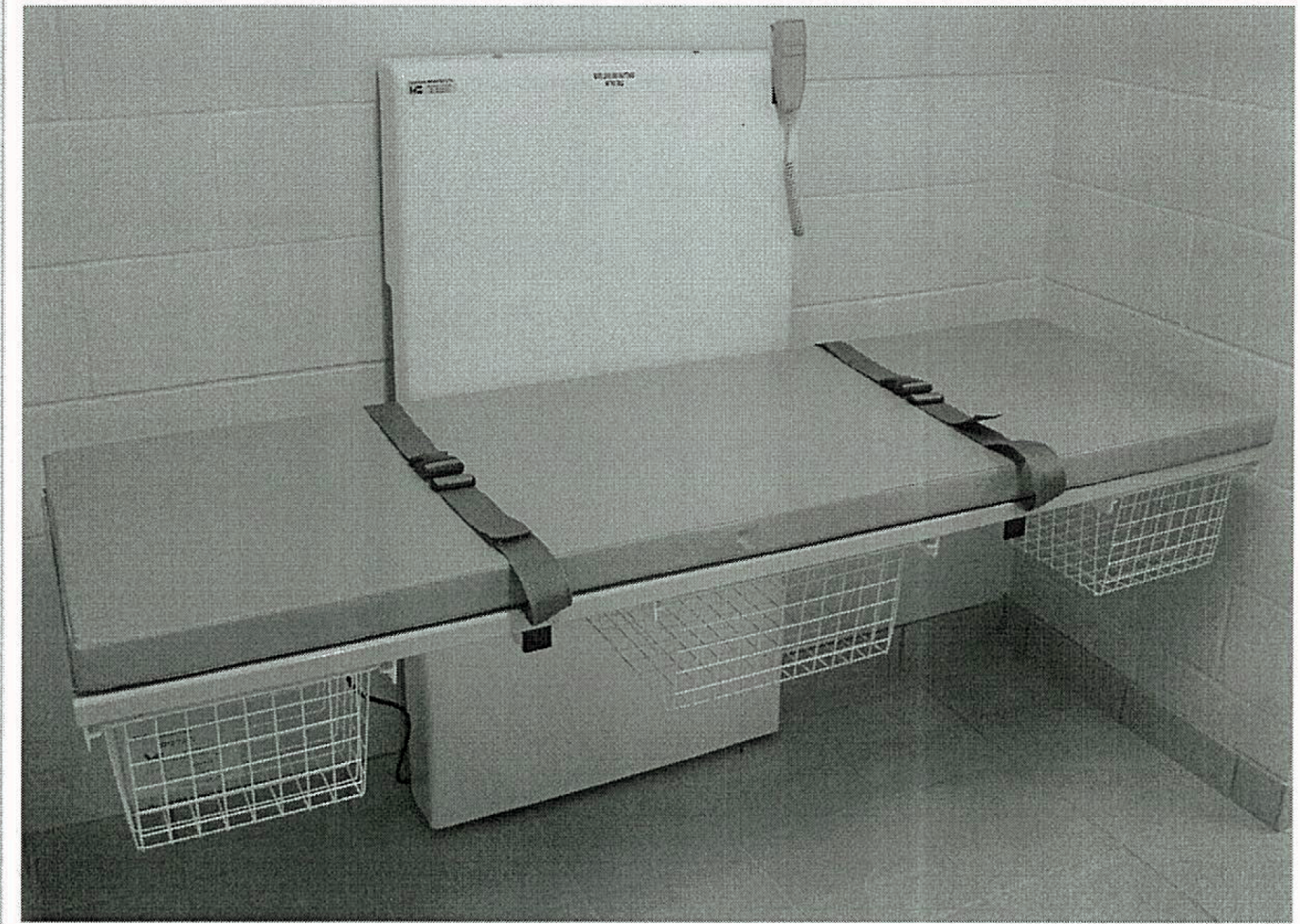

Model #	Size
19-88-62	62" x 24"
19-88-72	72" x 24"

Specific Features

- The padded top comes complete with two (2) integrated safety belts
- Three (3) wire baskets located under the table top are standard
- Adjustable in height from 21 inches (floor to padded) to 37 inches
- Powered by 110VAC to 24VDC power supply
- Electric hand control can be fastened to front cover with Velcro when not in use
- Front cover is acrylic for easy cleaning

BE GOOD TO YOUR BACK – WHEN HELPING OTHERS

Manufactured in Canada by Can-Dan Rehattec Ltd.

Installation Instruction

For Block Wall

- Place MAIN frame against the wall
(See Manufacture Specifications for complete instructions)
- Adjust the legs to get frame in level (use a level)
- Hold frame against the wall and predrill with 3/8" drill and install "Single Expansion Shields"
- Bring the frame back and fasten with 3/8" bolts

Power Supply

Place front cover up close to frame and connect plug from front cover to the power supply box
Note: In slot on box

Front Cover

Place front cover over MAIN frame and fasten with ¼" bolt

Operating and Handling Instructions

- Using the hand control the care-giver lowers the table to a height where they can easily and safely transfer individual from wheelchair to padded table top.
- Upon completion of transfer, care-giver must secure individual using the 2 (two) integrated body safety belts provided.
- Once individual is safely secured to change table, care-giver can proceed to raise table to a comfortable working position.

Before using always make certain:

- The Instruction Guide has been carefully studied and understood.
- Care-giver has received appropriate instruction and training.
- Individual is secured to the table with the 2 integrated safety belts.
- At any time **DO NOT** store anything under the table top.

*Requires a GFI Receptacle (15 amp)

NEVER LEAVE USER UNATTENDED ON TABLE

Maintenance

Check the following regularly:

- Check safety belts for signs of wear
- Clean Velcro on table top and mattress

How to remove Front Acrylic cover

Remove table top

Unscrew two ¼" bolts on top of front cover

Unplug the hand control from the power supply box

Replacement of Hand Control

Follow instruction above how to remove front cover

Remove two (2) screws on the side of front cover (top and bottom)

Lubrication of Wall Mounted Frame

One (1) time each year, REMOVE the front cover (follow instructions above) and grease the steel track. Only use "Super Lube" Synthetic Grease with Syncolon.

Check list for yearly maintenance

Date performed: _____

- Safety belts
- Clean Velcro on table top and mattress
- Lubricate track (follow instruction above)

***Copy this sheet and keep for your file**

Trouble - Shooting

If the table fails to operate....

Check the following:

- The table is plugged into the AC outlet properly
- There is power to the AC outlet
- Green light located on power supply is lit (indicating power from your AC outlet)

If the hand control fails to operate....

Check the following:

- Make certain that the hand control is plugged into the power supply box properly.

- ❖ For other service related problems, contact your local Can-Dan Rehatec Ltd. representative.

Description of WARRANTY

Can-Dan Rehatec Ltd. as manufacturer warrants from the **DATE OF DELIVERY TO THE FIRST CONSUMER** that each item described below, as **NEW AND UNUSED**, will be free from any defects in material and/or workmanship for a period of:

Two (2) years on Change Table Frame.

(Manufactured by Can-Dan Rehatec Ltd.)

1 (one) year on Linak Actuator

Six (6) consecutive months for accessories

Accessories include: Hand Control
 Table Top
 Mattress
 Safety Belts
 Front Cover

Defective parts must be returned to: **Can-Dan Rehatec Ltd.**

All parts replaced under this warranty become the property of:

Can-Dan Rehatec Ltd. Ancaster, Ontario, Canada

The warranty specifically excludes:

Normal wear and tear items.

Labour, parts and lubricant costs of all maintenance services.

Installation done by a non-authorized representative.

Damage resulting from improper repairs, modification or use of non- approved parts or, repairs done by a non-authorized **Can-Dan Rehatec** representative.

Damage resulting from accident, fire, theft, vandalism or any Act of God.

Power Supply No: _____ Serial No: _____

Signature: _____ Date: _____

Manufactured by

CAN-DAN REHATEC LTD.

3-1378 Sandhill Drive

Ancaster, ON L9G 4V5

Phone: (905) 648-7522

Fax: (905) 648-7799

Dealer

