en	Urinary leg bags, extension tubing, and leg bag straps
da	Urinbenposer, forlængerslange og stropper til benpose
de	Urinbeutel, Verlängerungsschläuche und Beinbeutelgürtel
es	Bolsas de orina de pierna, tubo extensor y ligas para bolsas de pierna
fi	Reisipussi, jatkoletku ja reisipussin hihnat
fr	Poches urinaires de jambe, tubulure d'extension et lanières pour poche de jambe
fr-GB	Sacs à jambe pour la collecte d'urine, tube de rallonge et courroies de fixation du sac
it	Sacche urinarie da gamba, tubo di prolunga e cinghie per sacca da gamba
ja	レッグバッグ、チューブ、レッグバッグストラップ
nl	Beenzakken voor urine, verlengslangen en bandjes voor de beenzakken
sv	Urinbenpåsår, förlängningsslangar och benpåsremmar

en INSTRUCTIONS FOR USE

Contents (See product label)

- Urinary leg bag with drain valve
- Extension tubing with connector
- Urinary leg bag straps

Product Description: Urinary leg bag designed to be emptied or to connect to a night drainage bag by extension tubing. Straps to hold a urinary leg bag securely to the thigh or calf. Not made with natural rubber latex. Product is made of polymer/copolymer plastics.

Indications and Intended Use: A urinary leg bag collects and contains urine. The extension tubing diverts urine to urinary collection bags. The straps secure a urinary leg bag to the leg. These accessory products are for individuals with urinary incontinence or a urinary stoma.

Non-sterile products

WARNINGS

- Disinfection and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

Precautions

- Single patient – multiple use. Sharing this product with multiple users creates risk of infection.


Sterile products

WARNINGS: To help reduce the potential for infection and/or other complications, do not reuse.

Notes: Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Reg or local distributor.


Applying the leg bag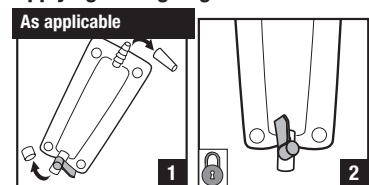

Notes: Non-sterile products

Between uses, empty and rinse leg bag with lukewarm water or clean per healthcare provider recommendations. Hang to dry away from heat. Wipe outside of bag and tubing with damp cloth as needed. Clean the connector with an alcohol wipe before each use. Discard when visibly damaged or stained.

Applying the extension tubing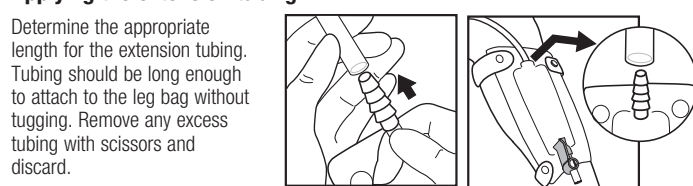


Attach the free end of the extension tubing to the connector at the top of the leg bag. Make sure tubing is not kinked or obstructed.

Applying the leg bag straps


The strap's ribs must be worn next to the skin in order to keep the bag stable. Adjust the straps so they are comfortable and secure.

Precaution


- To ensure adequate circulation, do not overtighten. Check the area under the straps regularly for skin irritation or trauma.
- To prevent improper drainage do not attach leg bag to knee.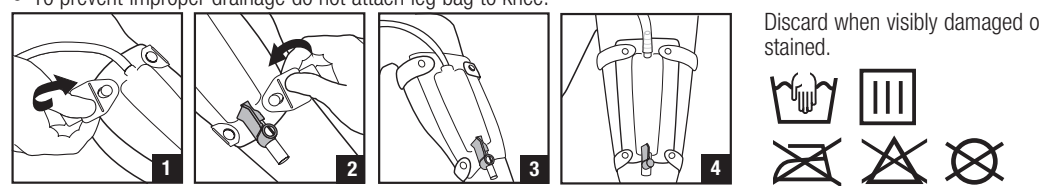

Draining the leg bag

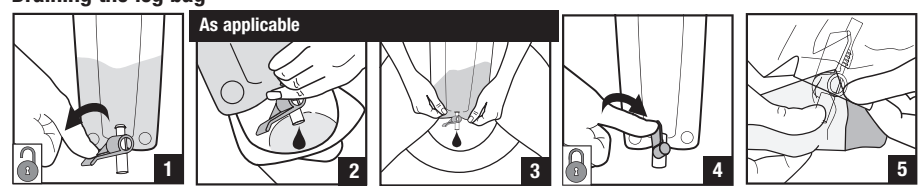To remove and dispose (Please follow all local disposal regulations)

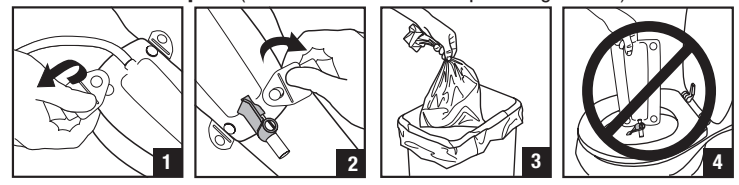Refer to product labelling

Non-sterile products	Sterile products	
		CE mark as required
		www.hollister.com/symbolglossary
		Consult instructions for use if sterile package is damaged
		Single sterile barrier system

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