




Figure 1

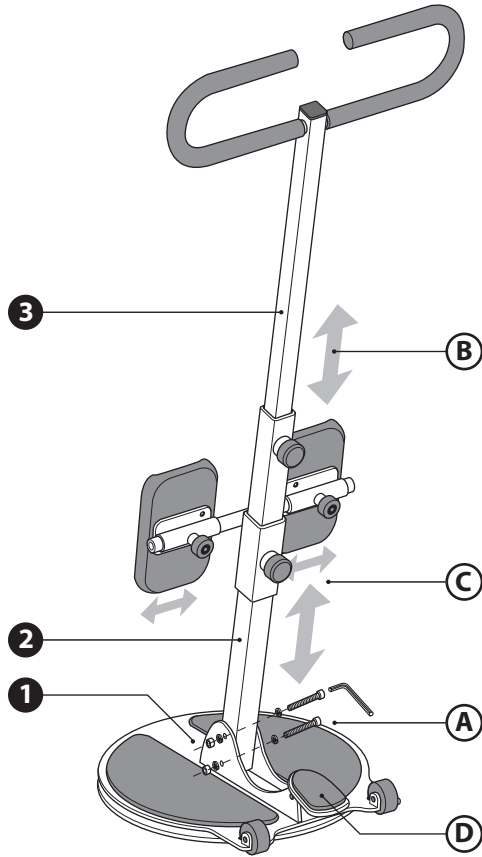

Assembly Instructions (See Figure 1):

Remove bolt securing main tube (2) to base (1).
Align base (1) and main tube (2). Assemble with bolt (A).

Instructions For Use (See Figure 2, 3 & 4):

Safe and easy patient transfers from one seated position to another seated position with the E-Z Turn Disc:

- Adjust the height of the handle (3) to suit the patient and caregiver (B).
- Position the E-Z Turn Disc between the seated patient and the other seated location.
- Adjust the height and width of the knee pads (C). Position the patient's knees centrally on the knee pads.
- The caregiver locks the brake pedal (D) and securely holds the upper handle bars.
- The patient grabs the lower handle bar and pulls themselves to an upright position (Figure 2).
- While the patient securely holds the lower handle bar, the caregiver releases the brake, rotates/swivels the E-Z Turn Disc (Figure 3).
- The caregiver locks the brake and the patient lowers themselves to the new seated position (Figure 4).



Built-in wheels make the E-Z Turn portable and maneuverable, but are not intended for moving patients during transfer.

Figure 2



Figure 3

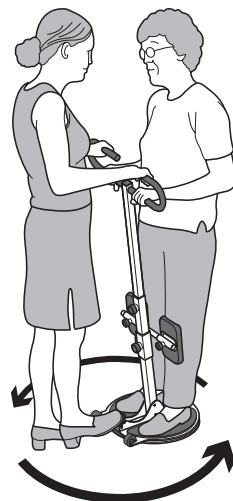


Figure 4

