

Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



Toilet Safety Frame

300 lbs weight capacity



MHSTSF



844604088039



MOBB Health Care Ltd.

116A Viceroy Rd. unit 5-6

Concord, ON L4K 2M1

www.mobbhhc.com

Assembly Instructions

TOILET SAFETY FRAME SET UP & USAGE

1. Remove all parts from the box: one bracket, two arm rest frames and two adjustment legs.

Height Adjustable Legs



Bracket



Arm Rest Frames

2. Remove the existing toilet seat and lid from toilet.

3. Place the toilet safety frame bracket between the toilet opening and the tank with the rounded part of the bracket closest to the tank.

See Fig. A. The holes on the bracket should line up with the mounting holes on the toilet.

4. Replace the toilet seat and lid and reinsert screws through the bracket and then through the toilet.

5. Attach the adjustment legs to each arm frame. Push in on the spring loaded locking pin and slide the adjustment leg onto the frame until the push pin engages into the first hole. There should be an audible "click" when the push pin engages.

6. Place the frames next to the toilet and turn the small "L" shaped tube towards the toilet until the push pins engage into the arms.

7. Remove star knobs and insert side frames into the bracket. Adjust to the width that best fits your space. Resinsert start knobs to lock side frames in place.

8. Adjust the height of the legs by depressing the push pin and sliding the leg up or down to the desired height. Make sure both legs are fully engaged and adjusted to equal heights. (Arm rests should be parallel to the ground).

Fig A.



CLEANING AND CARE

Regular cleaning is recommended. Use a non abrasive cleaner or detergent. Rinse with warm water and dry completely. Abrasive cloths may damage the surface. It's recommended that you regularly inspect your Toilet Safety Frame and it's parts for any corrosion, breakage, compression or wear.

